

DIAGNOVITAL MAGICPREP FAST EXTRACTION KIT 2

For in vitro diagnostic use. For professional use only





09090002 100 Tests



RTA LABORATUVARLARI BİYOLOJİK ÜRÜNLER İLAÇ VE MAKİNE SAN. TİC. A.Ş. GEPOSB Cumhuriyet Cad. No:3 41400 Gebze / Kocaeli / Turkey Tel: +90 262 648 5300 E-mail: rta@rtalabs.com.tr Web: www.rtalabs.com.tr



INTENDED USE

DIAGNOTransfer Tube contains 3 mL of viral nucleic acid extractive and preservative liquid. When clinical specimens suspected of respiratory tract infection are transferred in this tube, the liquid inside the tube can be used directly in real-time PCR (qPCR) reactions. The nucleic acid extractive and protective liquid inactivates all viral, bacterial or eukaryotic pathogens in the sample 5 minutes after contact with the clinical specimen.

MATERIALS PROVIDED

Kit Content	Amount / Qty	Storage	Transfer	
DIAGNOTransfer	100 Tubes	Room Temp.	Room Temp	
Tube			_	
Nylon-flocked /				
Rayon / Dacron	100 Swabs	Room Temp.	Room Temp	
Nasopharyngeal			_	
Swab				
Materials supplied by the user				
-Vortex				
-100-1000 μl Micropipette tips (DNase and RNase free)				

STORAGE

• Store all components room temperature.

• If the kit components have been damaged during transport, contact RTA Laboratories. Do not use as performance may be compromised.

- Keep reagents separate from specimens to avoid contamination.
- Do not use after the designated expiry date.

SAMPLE PROTOCOL

Swab samples:

1. Place the swab into DIAGNOT ransfer Tube after the sampling.

Samples in liquid form

1. After vortexing the sample tube for 15 seconds, transfer 2 ml liquid sample into DIAGNOTransfer Tube.

Sputum

- 1. Mix and homogenize equal amount of sputum and nuclease-free water (VTM or 1x PBS can be used instead of water)
- 2. Transfer 2 ml liquid homogenized sputum into DIAGNOTransfer Tube.



RTA LABORATUVARLARI BİYOLOJİK ÜRÜNLER ILAÇ VE MAKİNE SAN. TİC. A.Ş. GEPOSB Cumhuriyet Cad. No:3 41400 Gebze / Kocaeli / Turkey Tel: +90 262 648 5300

E-mail: rta@rtalabs.com.tr Web: www.rtalabs.com.tr



TRANSPORTATION, ANALYSIS AND STORAGE PROTOCOL

1. Deliver DIAGNOT ransfer Tube containing the sample to the laboratory at 2-8 $^{\circ}$ C within 3 days at the latest.

2. Vortex the tube at the highest speed for 15 seconds.

3. The sample in the tube is ready to use in qPCR.

4. If the samples are not subjected to qPCR analysis immediately, store them for no more than 5 days at 2-8 $^{\circ}$ C. If further delay is expected, samples can be stored for 1 year at -20 $^{\circ}$ C.



Symbols

2	Expiry Date
LOT	Lot/ Batch
REF	Catalog number
X	Temperature Limitation
\wedge	Caution
	Manufacturer In Vitro Diagnostic Medical Device
Ĩ	Consult instructions for use or consult electronic instructions for use

 ∇

Contains sufficient for (n) amount tests